ART. 4CS.445.xxxx.xx

Users manual Rev.4 2020-09

MPC.IMB.ISTRUZ.116

SOSTEGNO DI SICUREZZA RIBALTABILE / LIFT-UP SAFETY SUPPORT / BARRE DE SÉCURITÉ RABATTABLE / STUTZKLAPPGRIF

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EN INSTRUCTIONS

Read before installation, use and maintenance. This document is an integral part of the product: please keep it. All sizes are in mm. If you cannot read this document, it is possible to download this documentation on our website www.pba.it.

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EN
This product is comply with the directive 93/42/EEC for medical device.

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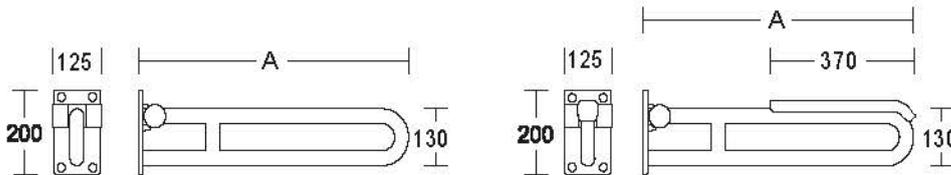
EN INTENDED USE
The lift-up supports are aids for disable or aged people and are intended to be used by an adult. The lift-up safety supports are wall-mounted, especially in bathrooms, on both sides of the sink and the toilet. The lift-up safety supports offer a rest and support aid and are used by people with physical impairments to help them stand up safely from the seated position, with or without the aid of an assistant.

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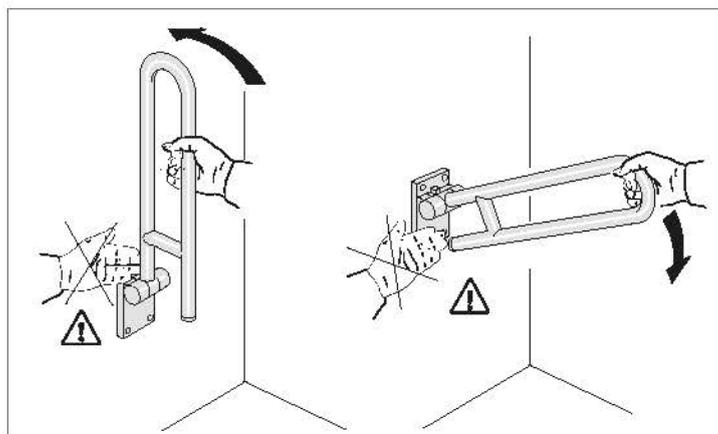
EN - APPLICATIONS AREA

Hospitals, rehabilitation centres, nursing home for elderly, public buildings, private bathrooms.

Movimentare e utilizzare l'ausilio come da immagine: attenzione schiacciamento dita. Move and use the safety support as shown in the picture: attention to squeezing fingers. Déplacer et utiliser l'aide comme indiqué sur l'image: attention aux doigts écrasés. Bewegen und verwenden Sie Stützklappgriffe wie im Bild angegeben: Achtung vorm Quetschen der Finger.

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EN SAFETY PRECAUTIONS AND INDICATIONS FOR MAINTENANCE

WARNING: All disabled accessories must be fixed in accordance with the table of fixing materials. pba S.p.A. is not liable for any damage/loss of functionality of the product caused by not observing the recommendations set out in our instruction manual. Under NO circumstances must the pba S.p.A. disabled and bathroom accessories be altered, tampered with or modified. Non-observance of these instructions makes product guarantees null and void.

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EN

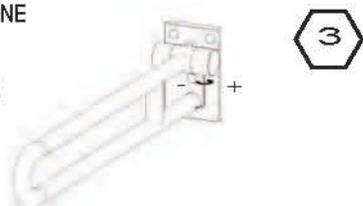
Do not use the lift-up supports improperly or for purposes other than that indicated in the instruction manual. Avoid installing the supports inside saunas or inside too much hot rooms (a temperature of the support's surface exceeding 41°C could cause skin damages). If the support doesn't work properly, contact pba technical office (TEL +039 0424 5451) or pba Deutschland GmbH (+049 2841 99890 0). On a periodic basis (at least once a month), make sure the support lowers slowly and smoothly: if necessary, to increase the braking effect of the clutch, simply adjust the actual clutch following the instructions written below. Also make sure, on a periodic basis, that the wall-mounting bracket is not damaged or has not worked loose. The application of a load heavier than the maximum weight indicated could permanently damage the lift-up safety support and its wall-mounting bracket. If this should occur, and also if a permanent deformation of the support is visible, contact pba SpA (TEL +039 0424 5451) or pba Deutschland GmbH (+049 2841 99890 0). Before lowering the support, make sure nobody is near (where it moves). The failed observance of these instructions could cause serious damages and injuries. **WARNING:** Please do not leave your child unattended. There is a risk of injury. The supervision of an adult is required.

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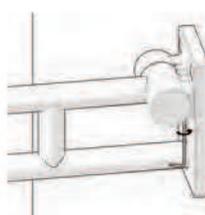
Chemical compositions of AISI stainless steel grades

Material - Stainless	C%	Cr%	Ni%	Mo%
AISI 430 (1.4016)	0,12 max	14,0-18,0	/	/
AISI 304 (1.4301)	0,08 max	18,0-20,0	8,0-10,5	/
AISI 316 (1.4401)	0,08 max	16,0-18,0	10,0-14,0	2,00-3,00
EN AISI 316L (1.4404)*	0,03 max	16,0-18,0	10,0-14,0	2,00-3,00

*Standard PBA

Estimated pit corrosion penetration time - time to penetrate 1 mm (years) by steel type

Material - Stainless	Marine	Semi_industrial	Rural
AISI 430 (1.4016)	N/A	85	250
AISI 304 (1.4301)	145	135	770
AISI 316 (1.4401)	290	525	1200

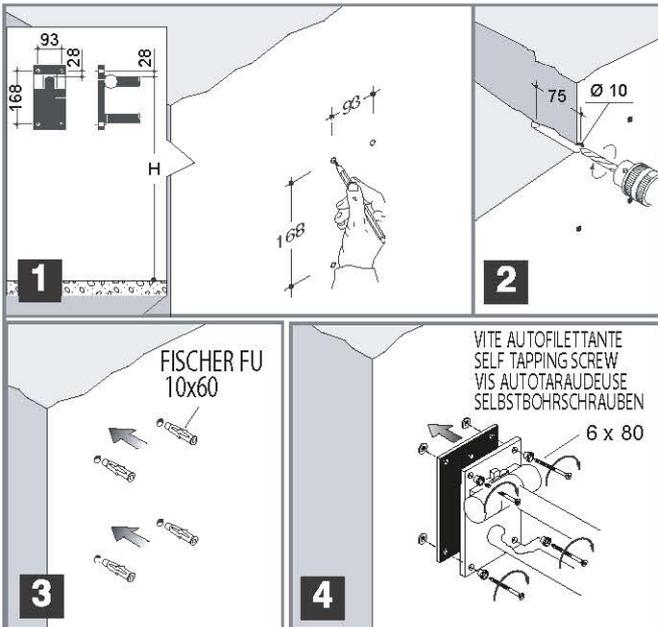
Source: The British Stainless Steel Association (BSSA)

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EN The AISI 316L stainless steel (1) steel used by pba, meets the requirements of UNI EN 1670, the ultimate quality standard for ensuring resistance to corrosion. Pba has been awarded a certificate for successfully salt spray testing the stainless steel in accordance with UNI ISO 9227. (1) X2CrNiMo 17-12-2 as per EN 10088-3

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EN
INSTALLATION PROCEDURES BY STANDARD FIXING EQUIPMENT

The following installation procedures refer to the fixing of disabled hardware to concrete. For other types of building construction, refer to the table of fixing material.

⚠ Before installing the item, write down the serial number on the logbook provided.

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EN

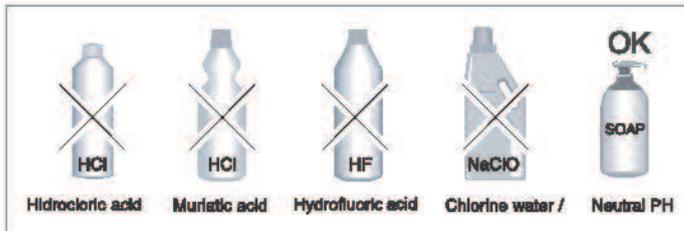
NOTE: for fixing to other materials contact the pba technical office.

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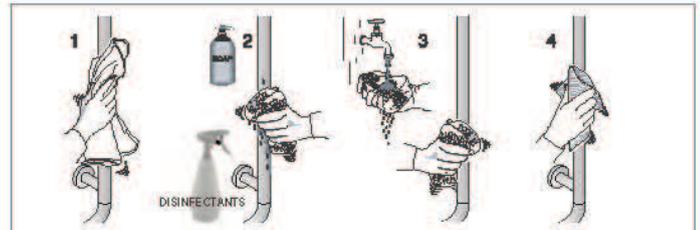
EN

INDICATIONS FOR CLEANING

The products only require simple cleaning and maintenance.

To clean pba products periodically use a wet cloth. As an alternative use neutral detergents or disinfectants diluted in a solution having neutral PH. After washing rinse the products with cool water and let them dry. Always select the correct soaps/disinfectants in accordance with the manufacturer's instructions (ensuring that they are compatible with the different surfaces and correct concentration). It is recommended to test the detergent/disinfectant on a small hidden area of the product before use. Wash the part in the direction of the satin finish to avoid scratching the surfaces. The use of detergents containing chlorine, acids and alkali, organic solvents and abrasive substances is not recommended. Do not use any abrasive objects (like brushes) for the cleaning.

Furthermore, avoid using abrasive sponges or steel wool, as these will contaminate the surfaces of the material. Finally, stubborn deposits on the surfaces of the part must be removed using a wood spatula (not a steel one), to avoid scratching and contamination. Pba S.p.A. declines all responsibility for damages caused by use of unsuitable cleaning methods listed above.

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EN**INFORMATION FOR THE DISPOSAL OF WASTE PRODUCTS**

At the end of its life, the product must be disposed according the local waste regulation. The article is made of AISI 316L stainless steel and recyclable polyamide.

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EN

Under the intended conditions of use and in accordance with the guidelines for maintenance and cleaning, the expected life time of the product is of 5 years.

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WALL RESISTENCE

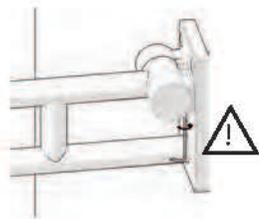
The wall must have sufficient strength to ensure the application of 250 kg at each fixing point.

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MAINTENANCE

- 1) Check that the support is well anchored to the wall.
- 2) If the lift-up support does not remain in a vertical position, tighten the specific grub screw positioned under the shell.
- 3) In case of breakage of any component or in case of anomalies, contact immediately the manufacturer and do not use the support.
- 4) It is recommended to carry out maintenance checks every 1 year. Record the checks carried out in the appropriate attached form.

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